



Drench System

Innovation Award Winner

Pack and waist-belt for application of oral and pour-on animal health products

- Comfortable harness with air mesh padding
- Suits 5 and 2.5L containers
- Top or bottom draw-off
- Includes waist belt with gun holster
- Retainer clips to protect draw-off hose



**DESIGNED BY FARMERS
FOR ARMERS**



Also available Vaxiholster for injection bottles

Instructions

To get the best out of your pack, it is preferable to have a 130cm draw-off hose or longer. This allows you to clip it into both the holders on the side of the pack.

Preventing problems with animals getting caught in the hose.

Fill your container, screw on the cap and then feed the gun thought the bottom of the pack. Most guns will fit through the opening at the bottom. Alternatively put the container in the pack to fill.

Use the adjusters on the side of the pack to make the bottle a snug fit.

To attach the belt to the pack, thread the pack tail through the adjuster in the middle of the belt. Then set the belt to suit your height.

The gun holster is fitted by forcing the tapered end through the large loop on the belt and can be fitted to suit left or right handers.

Alternatively the gun holster can be threaded through the belt by removing the side release buckle and threading it through the webbing, sitting it behind the belt keeper to hold it in place.

Ezepak Ltd
info@ezepak.co.nz
Phone +64 6 376 6273

